

# Diamond Dermabrasion Series




**MANUAL**

# NV-908 4 Functional Diamond Dermabrasion

## 4 Functions

- Diamond Dermabrasion
- Cold & Hot Treatment
- Ultrasonic
- Skin Scrubber

## Machine Structure

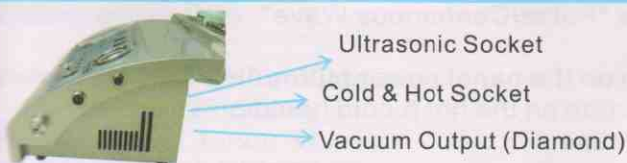

## Face Panel Instruction



**Contents/ Function & Treatment/ ①②③④**

**1- Finish all accessories connection before plug the power cord.** Don't plug the accessory power wires to the sockets when the machine powers are on.

**2- Be sure the Main Power Button on the machine right side and the panel button "POWER"(Power On/Off) are on→ Set the "TIMER"**

● **Diamond:** Press ON (Diamond ON/OFF) to start→ Adjust the Intensity Knob for the suction you want→ Press OFF (Diamond ON/OFF) after use.

● **Ultrasonic:** Press "Wrinkle" and adjust the INTENSITY→ Press >/■ to start, choose "Pulse/Continuous Wave"→ Press >/■ to stop after use.

● **Cold & Hot:** Turn on the panel power button "HOT & COLD"→ Press the power button on the hot & cold hammer→ Turn off both power buttons after use.

**3- ● Skin Scrubber:** Be sure the Main Power Button on the machine right side is on→ Set scrubber TIMER and adjust the scrubber INTENSITY→ Press scrubber ON/OFF to start, choose "Pulse/Continuous Wave"→ Press scrubber ON/OFF to stop after use.

**Caution:** Please don't turn on usage with long time, better use scrubber within 15~30 minutes per time. Then stop the scrubber for 20 minutes before the next scrubber operation. Long time operation will burn the scrubber grey box and destroy the control board of the whole machine.

**4- After Operating,** turn off the power buttons and disconnect the power cord, then disconnect and clean accessories.

## Package Included

Diamond Dermabrasion Machine X 1  
PVC vacuum hose 6\*4 X 1  
Stainless steel wands X 2  
Diamond tips X 9  
Cotton filter X 2 packages (1big, 1 small)  
Ultrasonic probe X 2 (flat and pointed)  
Cold/Hot Hammer X 1  
Skin Scrubber Handle X 1  
**Negative Current Handle X 1**  
Rubber rings and fuses  
Full detailed English manual/ instructions

## Specifications

**Voltage:** 220V - 240V/50Hz □  
100V - 120V/60Hz □

**Power:** 110W **Fuse:** 3A

**Packing Size:** 51.5\* 39\* 21cm

**G.W:** 11kg

# 1Diamond Microdermabrasion Function

## ※ Function

The Diamond Microdermabrasion provided a non-surgical skin refinish procedure, by using sterile diamond heads to abrade or rub off the top skin layer, then Vacuuming out the particles along with any dirt and dead skin back up. This procedure removes skin debris, imperfections, blemishes, wrinkles and unwanted pigmentation on the skin. Moderate usage of Diamond Microdermabrasion, accompany with skin products that penetrate through top skin layer that reach the dermis level, helps to replenish natural nutrients, restore cell activity, and promote skin health. The results after Diamond Microdermabrasion treatment is usually a healthy, glowing, beautiful skin.

## ※ Accessories



PVC hose  
×1



Rubber rings  
& Fuses



Big cotton filter  
×1 pack



Small cotton filter  
×1 pack



Color Box

Open →



Stainless steel wands ×2  
Diamond tips ×9

## ※Preparation before treatment

1. A consultation included the examination of the skin, a plan for treatment.
2. Operator should wear vinyl gloves and a facemask.
3. On oily or acne skin, a steamer may be used prior to the treatment, or a degreasing lotion may be applied prior to the cleanser.
4. Cleansing the skin with Cleansing gel or water, then wipe off with cotton pad. Allow the skin to dry.

## ※Operating Instruction

1- Please first connect all accessories to the machine.



1) Unscrew the vacuum connector, make sure a big cotton filter is in the center & a large black ring is around connector head



2) Screw down the connector after check. Then attach the PVC hose to the connector.



3) Place the other end of PVC hose in the end of wand.



4) Place a small cotton filter on the head of wand. (see, a small black ring is around the head)



5) Screw the diamond tip into the wand (choose proper size tip for customer, see behind **diagram of tips set**)

2- Put the plug into the power socket and turn on the power button. Turn the intensity knob to left side (minimum).

3- Adjust the vacuum intensity slowly from minimum to maximum, the vacuum meter will show the data of pressure. For commonly use, do not set the intensity to maximum unless necessary.

### Recommended Vacuum intensity on treatment area

Treatment Area	Normal Skin (inHg)	Sensitive Skin (inHg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10

